Hörverstärkende Telefon-Headsets
Amplifying telephone headsets

PL-100 / PL-200

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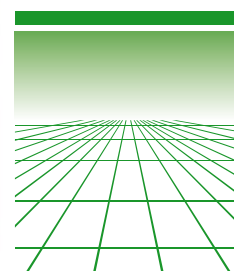
**No stress on
the phone –
understand every
word clearly**

**PL-100: Telephone headset
with neckloop**

**PL-200: Telephone headset
with headphone connection or
for direct connection to hearing
aids and CI-systems**

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PL-100 and PL-200 can be discretely attached to your clothes. Each amplifier comes with a built-in microphone which offers the function of a fully-fledged headset for mobile phones as well as stationary and cordless phones – e.g. in the office.

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Accessories offered by Humantechnik: various different headset designs and audio cables.

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PL-100 and PL-200 each have a 2,5 mm phone jack for the connection to telephones with corresponding jack plug (universal configuration)*.

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PL-100: Amplifying telephone headset with neckloop for »inductive hearing«

The amplifying PL-100 with neckloop produces an induction loop signal that can be received by a hearing aid with a built-in teleloop. This function is activated by setting the hearing aid to position »T« or »MT«.

The induction loop signal can be smoothly increased up to a volume of approx. 35 dB by turning the volume control.

PL-200: Amplifying telephone headset with headphone connection or for the direct connection to hearing aids and CI-systems

To the audio outlet of the PL-200 the user can connect various hearing aid devices – e.g. a headset – or an audio cable (monaural/binaural) via which the mobile phone is directly connected to a hearing aid or CI-systems which are equipped with the respective functions.

The volume can be adjusted up to approx. 25 dB using a variable volume control.

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HUMANTECHNIK GmbH